



# THE SOLUTION FOR MALE IMPOTENCE

## ZSI 475

### HYDRAULIC PENILE IMPLANT

(3 COMPONENTS)



## ALL STANDARD SIZES IN ONE BOX

From 12 cm to 24 cm and more (with extensions)

More informations at: [www.zephyr-si.com](http://www.zephyr-si.com)

ZEPHYR Surgical implants is ISO 13485 : 2004  
Hydraulic Penile Implant ZSI 475 is EC certified.

LOCAL DISTRIBUTOR

**ZEPHYR Surgical Implants**

Route des Jeunes 4  
1227 Acacias (Geneva)  
SWITZERLAND  
Email: [contact@zephyr-si.com](mailto:contact@zephyr-si.com) / [www.zephyr-si.com](http://www.zephyr-si.com)



Flyer, English version  
Version 8, November 2015



Swiss Made




# ALL STANDARD LENGTHS OF PENILE IMPLANT AND ACCESSORIES ARE AVAILABLE IN ONE BOX\*

\* Large Penis size part of 22 cm from 21 to 25 cm with extensions available in a separate box .




INTRODUCER WITH RULER

SPATULA

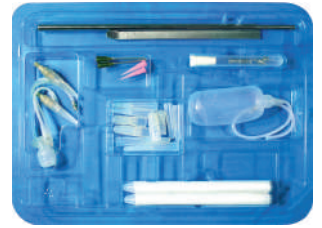

PROXIMAL PARTS

PUMP

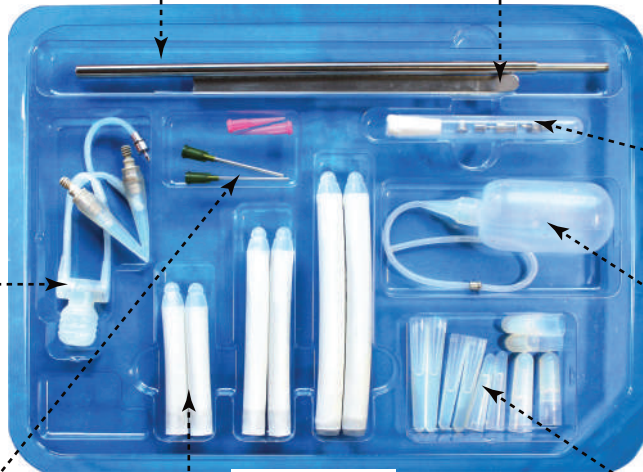

NEEDLES FOR INTRODUCER AND CONNECTOR

RESERVOIR 80 ml



NEEDLES FOR SOLUTION FILLING



Length 13 16 19 (cm)  
When Inflated Ø 16 16 18 (mm)

**DISTAL PARTS CAVERNOUS CORPUS PROSTHESIS**  
Ø RING 12.5 mm



cm 1 2 1 2 3  
DISTAL AND PROXIMAL EXTENSIONS

## ASSEMBLING



SELECT THE RIGHT DISTAL PART LENGTH



ADD EXTENSION IF NECESSARY




SCREW THE DISTAL PART TO THE PROXIMAL PART

Big Size Length 22 cm, Diameter 21 mm when inflated. Ø ring 13 mm.




PRESS THE BULB TO INFLATE THE PROSTHESIS  
THE DIAMETER INCREASES AS THE PROSTHESIS IS INFLATED


## FUNCTIONING

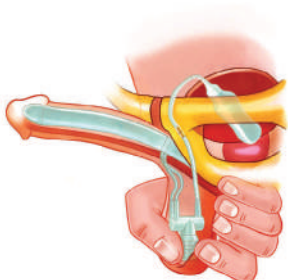


Press the bulb to inflate the prosthesis

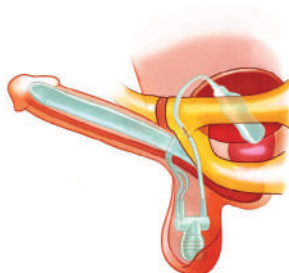

Penis erected

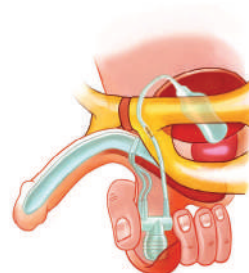

Press the cylinder to deflate the prosthesis

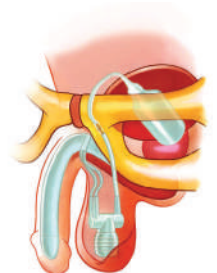

Penis flacid