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
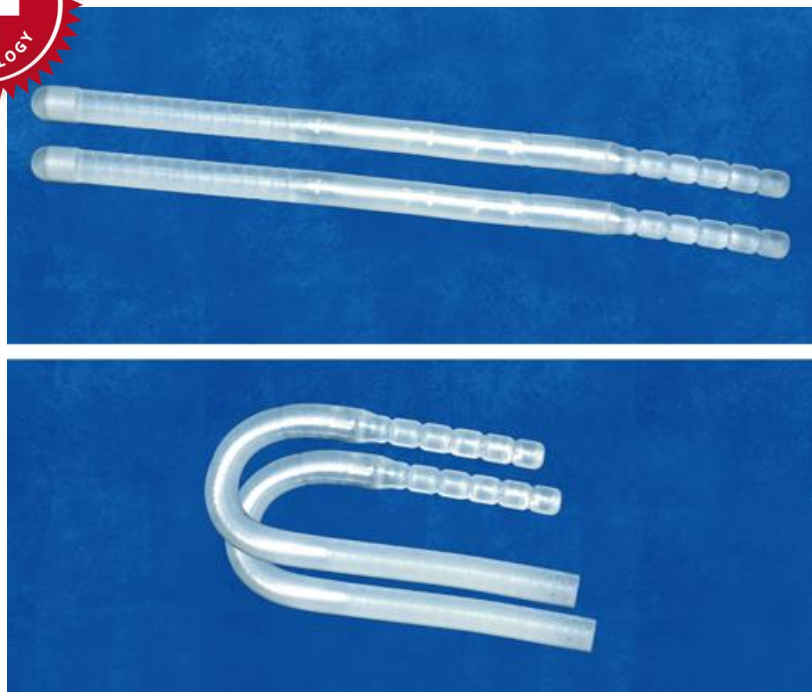
Functioning

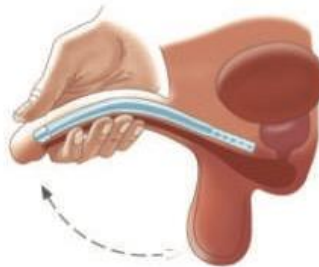
1. flacid penis

2. With a hand, straighten the penis from bent position.

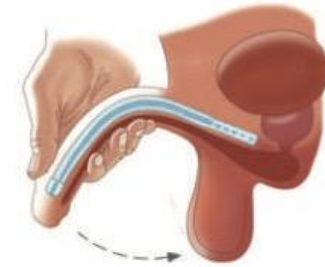3. Erect penis

4. With a hand, return the penis to bent position

Few infections, few mechanical failures

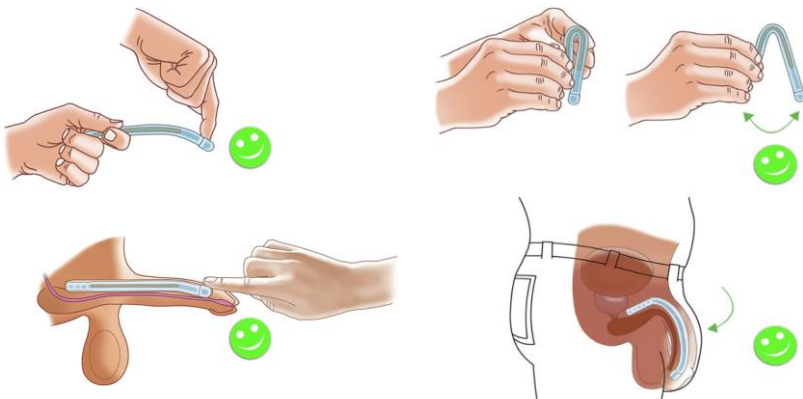
Made of silicone
With bendable Silver cable inside

SEMI-RIGID ZSI100


Indications:

- Patient with **organic erectile dysfunction and Peyronie's disease**
- Patient **unable to use the pump** of the inflatable penile implant
- Patient **requiring easy procedure (requiring short anaesthesia)**
- Patient **refusing risk** of adverse events or mechanical failure of inflatable penile implant
- Patient **unable to pay for an inflatable penile implant**

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
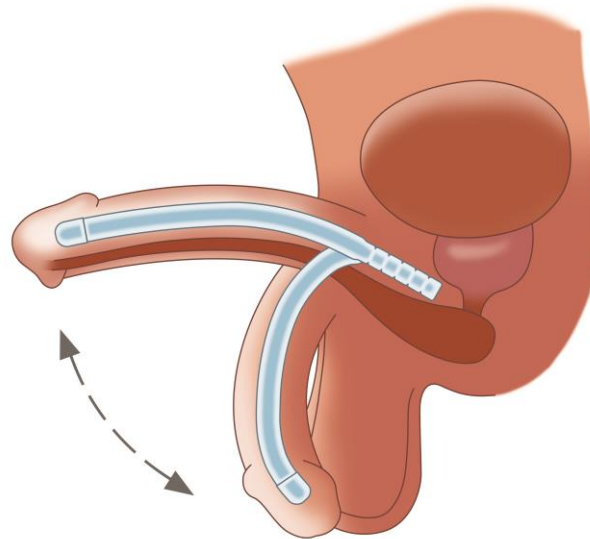
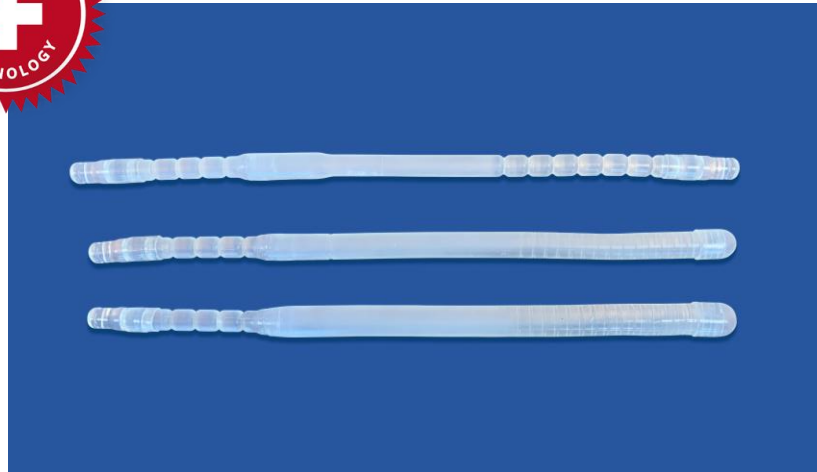
SOFT – Cable Free ZSI 100 CF


Indications:

1. After infection or erosion of an inflatable or semi-rigid penile implant
It keeps the space in CC

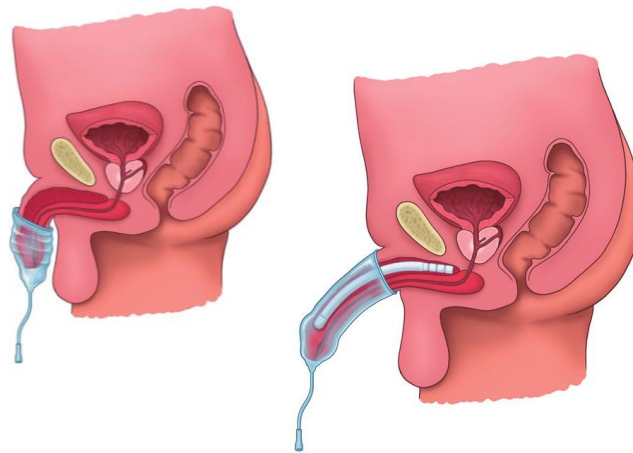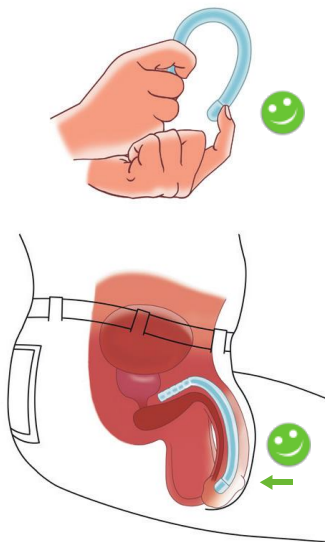2. Spinal Cord Injury or neurogenic patients with Erectile dysfunction
Lower risk of erosion and infection

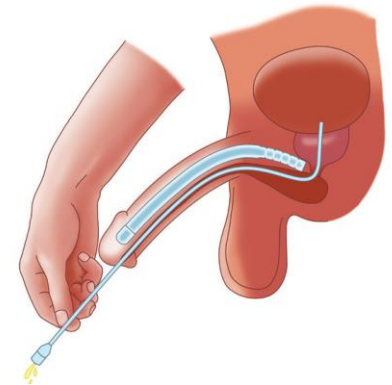
Help to maintain penile sheath
over the penis


Easier
catheterisation

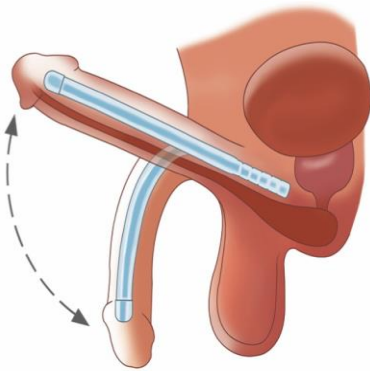
SOFT – Cable Free ZSI 100 CF

Flexible

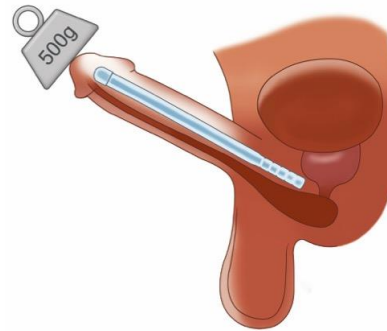
Possibility of sexual intercourse

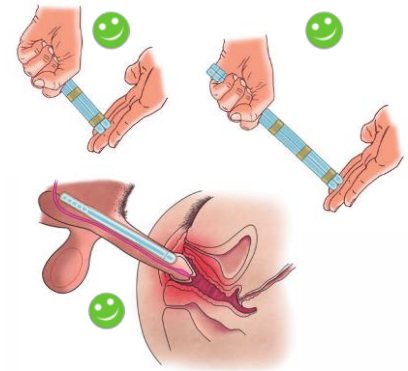
Few infections, few erosion, NO mechanical failures, more comfortable

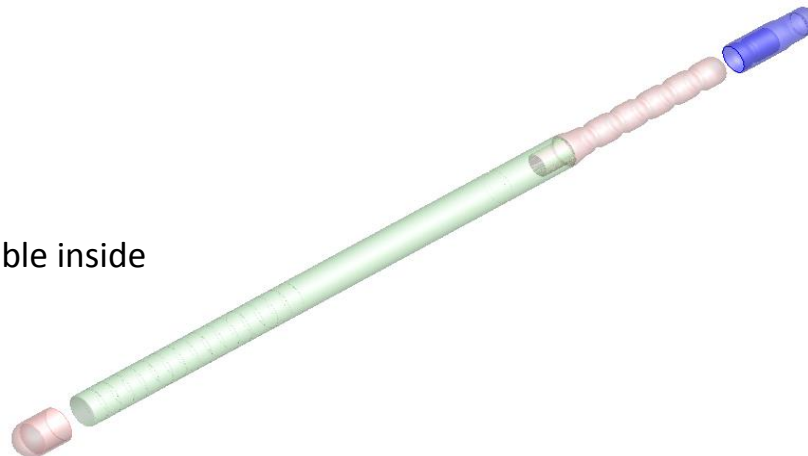Made of silicone
Flexible, without any cable inside

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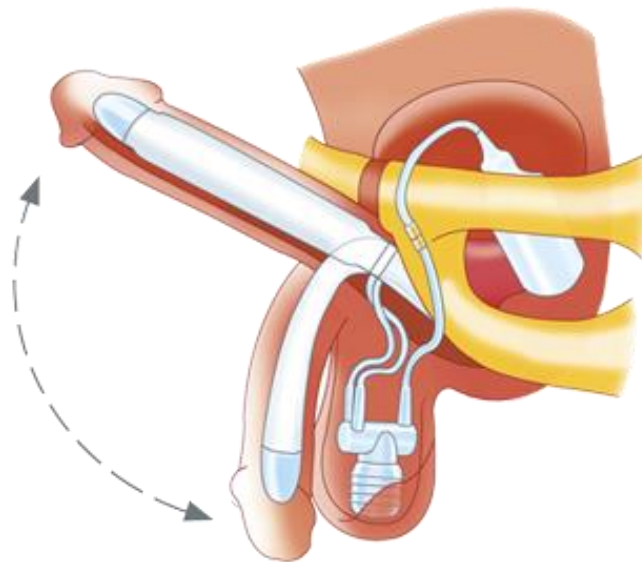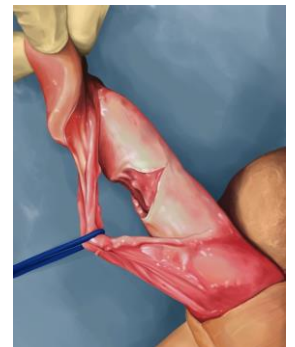
INFLATABLE ZSI 475


INDICATION

- **Patient with organic erectile dysfunction and Peyronie's disease**
- **Patient able to use the pump** of the inflatable penile implant
- **Patient accepting risk** of adverse event and mechanical failure
- **Patient able to pay** for an inflatable penile implant


INFLATABLE ZSI 475

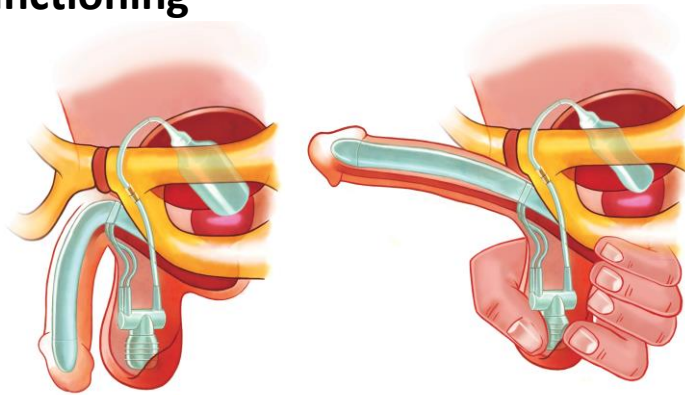
Functioning

Inflation

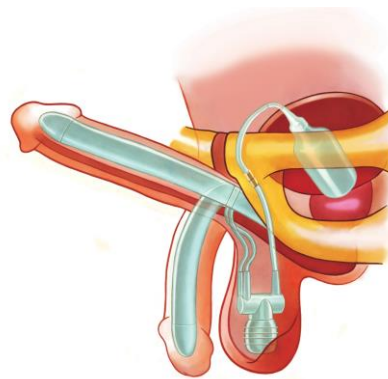Erection

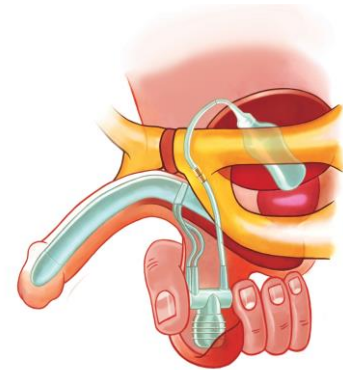Deflation

3-LAYER CYLINDERS

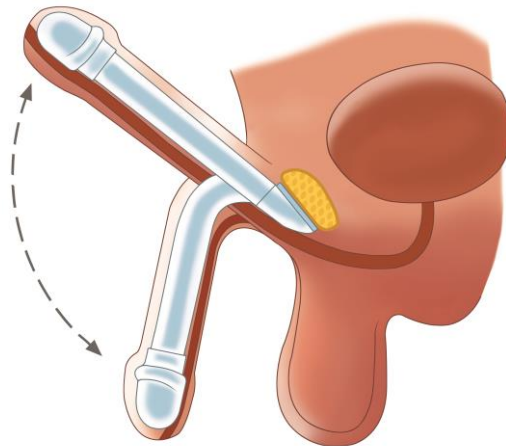
Bigger penile when implanted with **Better rigidity**

- Reducing the risk of piercing the inner layer
- Resistive against multiple folding
- Resistive against to strong pressures, (blowing resistance > 3.3bar)
- Low risk of buckling
- 100% BIOCOMPATIBLE SILICONE
- PVP hydrophilic coating (will absorb the antibiotic, the surgeon chooses to use)
- SAFE IN MRI AND AIRPORT SCANNERS

All standard lengths of penile implant & accessories are available in one box

ZSI 100 FTM MALLEABLE PENILE IMPLANT

ZSI 100 FTM MALLEABLE PENILE IMPLANT

ZSI offers solutions adapted for plastic surgery to satisfy the specific demands of transsexual / transgender men and women

Passage for urethra

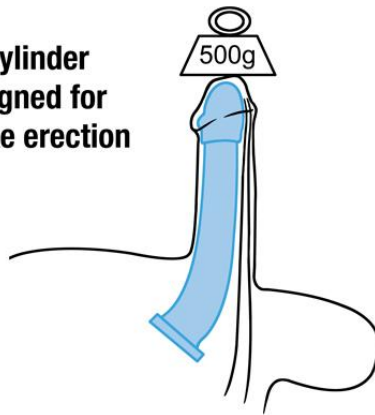
TOTAL LENGTH: 200 mm that can be reduce to 130 mm

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
**1 cylinder
designed for
life like erection**


ZSI 100 FTM MALLEABLE PENILE IMPLANT

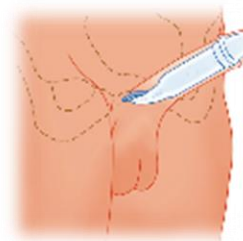Functioning

ZSI 475 FTM
Hydraulic penile implant

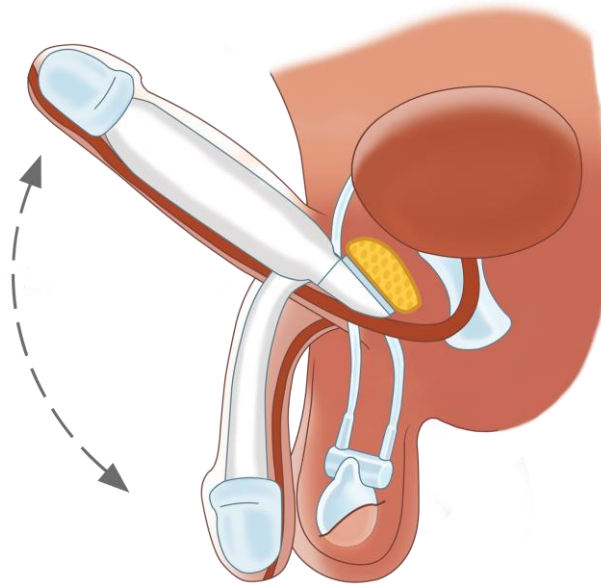
ZSI 475 FTM Hydraulic penile implant


Base with a square shape to fix the prosthesis to the pubic bone
One glans: smooth and large
Structure inside to reduce the fold in flaccid position
Tubing exit is in good direction to the scrotum
Glans look like a glans with a passage for urethra
Good angle in erected position
Large diameter of 22 mm

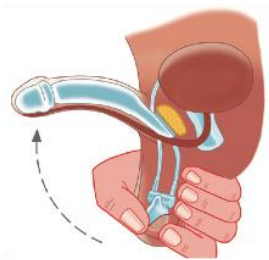
Functioning

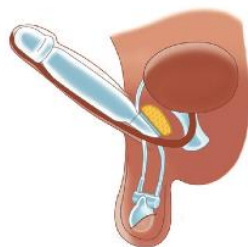1. Press the bulb to inflate the prosthesis.

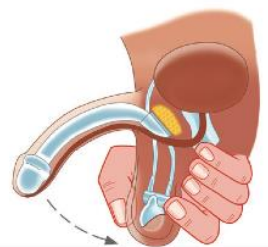2. Penis erected.

3. Press the cylinder to deflate the prosthesis.

4. Penis flaccid.

Thank you for your attention

Robina
Group Companies

Exclusive Agent
Tehran, Iran

+98 21 440 25 495

+98 21 440 25 207

www.robina-persia.ir
www.urologic.ir