

tensi+



NEW TREATMENT OF OVERACTIVE BLADDER



Uses the technique of **transcutaneous electrical nerve stimulation (TENS) of the posterior tibial nerve**. This technique is non-invasive and non-medicinal, and does not have any side effects.



Simple to fit



Discreet



Easy to use



Autonomous use

Indications



Urgenturia

The desire is irrepresible and urgent.



Pollakiuria

The desire to urinate exceeds 8 times per day.



Nycturia

The need to urinate occurs at least once per night.



Urge incontinence


It is impossible to hold it.

An easy to use-treatment



Step 1: Positioning the device

The strap is worn at the ankle. The bottom electrode sits close to the medial malleolus, slightly below and backward of this one.

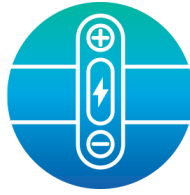

Step 2: Operating the device

A long press on the On / Off button is enough to start Tensi+. The stimulation intensity is gradually adjusted with the + button.



Step 3: A 20 min treatment

Tensi+ starts for 20 minutes and switches off automatically at the end of the treatment.




Tensi+ packaging

- 1 medical device Tensi+
- 2 electrodes
- 1 strap
- 1 USB cable
- Instructions

To add: 1 conductive gel



Success rate: 68% after 12 weeks.

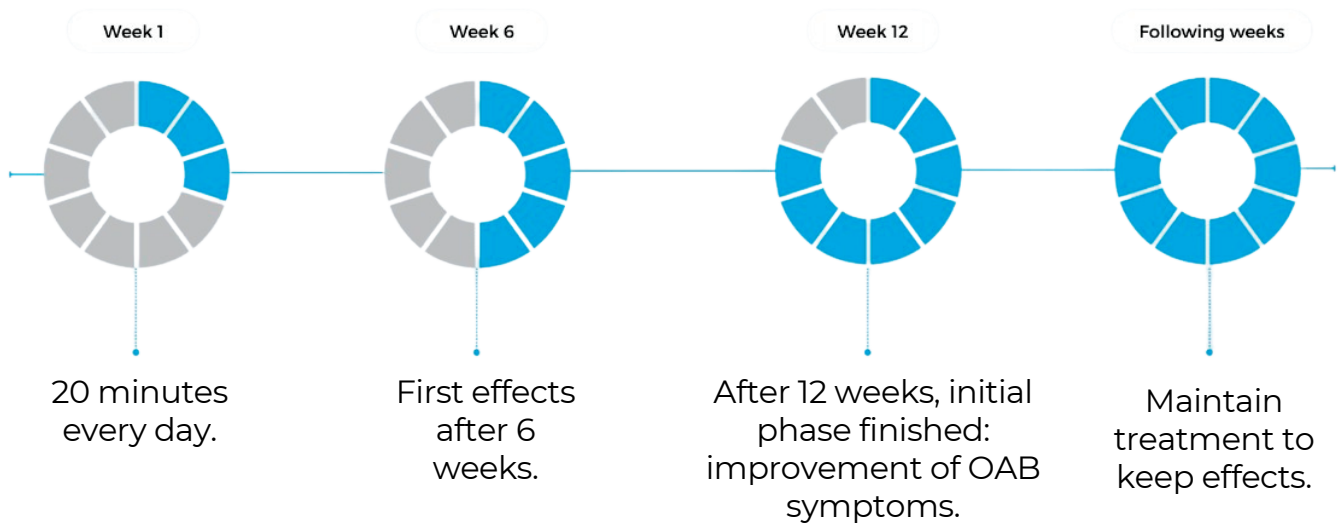

Stimuli
TECHNOLOGY

HEAD OFFICE
20bis rue Danjou 92100 Boulogne-Billancourt FRANCE


Robina
Group Companies

Telefax: (021) 440 25 207
Unit 07, No 05, Motahari St, Central Payambar St, Tehran, Iran
www.urologic.ir

